

How to use: Intelligent Warmer








Engineered for safety and maximum patient comfort.

1. Load product with the label face up.
2. Place pack fully in slot for proper sensor detection.
3. Remove products only after they display a green solid light, indicating it is fully heated.

Intelligent Warmer Status Indicator

What the lights mean:

-  **Heating** – Solid Red LED – Packs have not yet reached the required temperature.
-  **Heated** – Solid Green LED – Packs have reached the required temperature and are ready to use.
-  **Use First** – Green LED Flashes – Pack has been in the warmer for 84 hours. These packs should be used first.
-  **Discard** – Red LED Flashes – Pack has been in the warmer longer than 250 hours and should be discarded.

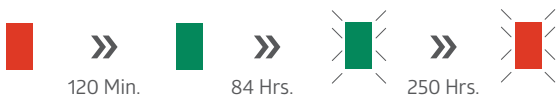

Intelligent Warmer Safety Tips

- DO NOT use commercial or alcohol based cleaners on the door. They will cause it to crack over time.
- DO NOT use chemical-based products to clean the unit.
- Keep exterior warmer sides clear of obstructions to allow for air intake and exhaust. A minimum of 3 inches is recommended.
- DO NOT place any items on top of the warmer.
- DO NOT place the front of the unit in direct sunlight, as this can impact sensor sensitivity.
- Read instructions for warnings, care and maintenance.



MSCWARMER28S
Warms up to 28 packs of wipes simultaneously

MSCWARMER14S
Warms up to 14 packs of wipes simultaneously